

Positioning Bed Pad

User guide

support@eyra.group

● SPECIFICATIONS:


-Fabric: nylon

-Dimensions: 48" x 40" inches

-Weight: 1.56 lbs

● HOW TO CLEAN:

Machine wash and air dry. No tumble dry or extended sun exposure. DO NOT use any fabric softeners or bleach.

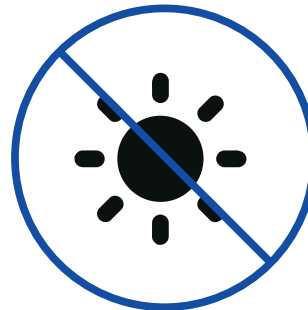


● INTRODUCTION:

Eyra positioning bed pad with handles is specially designed to help lift, transfer, turn and reposition bedridden patients. With positioning bed pad, taking care of patients will be easier, and unnecessary lumbar or back injuries can be avoided.

● NOTE:

1. You should help the patient lie in the middle of the positioning bed pad with his/her shoulder/scapula in line with the highest webbing to avoid body bending when turning the patient.

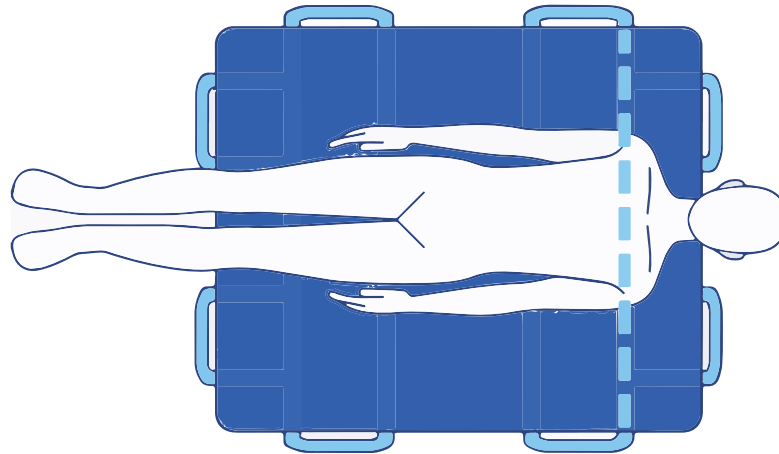


2. When lifting the upper part of patient body, please do it carefully and slowly, or else his/her neck may experience discomfort.

● HOW TO USE:


Step 1.

Help the patient lie on one side.

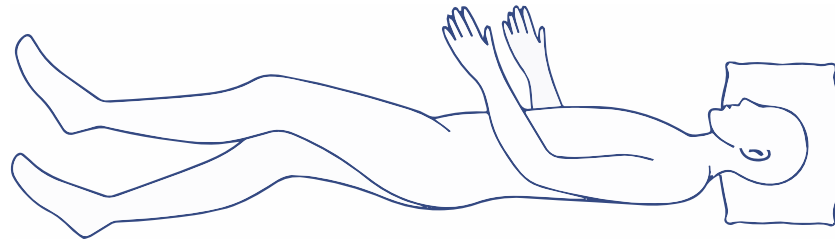



Step 2.

Open the positioning bed pad, roll up about 2/3 and carefully put it under the patient.

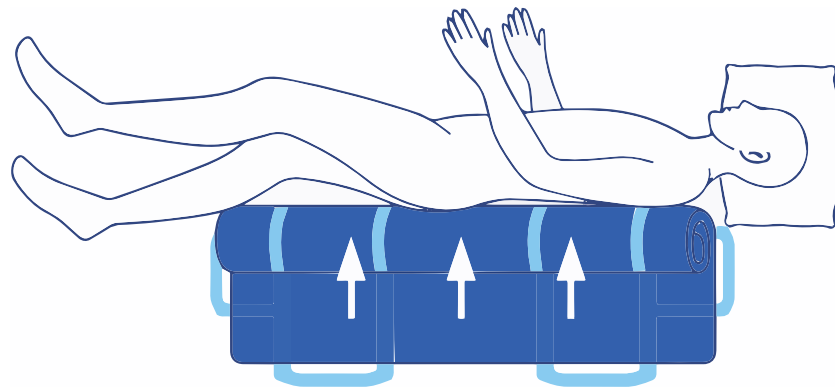


Step 3.

Help the patient turn and lie on another side and slowly draw out the rolled up part.

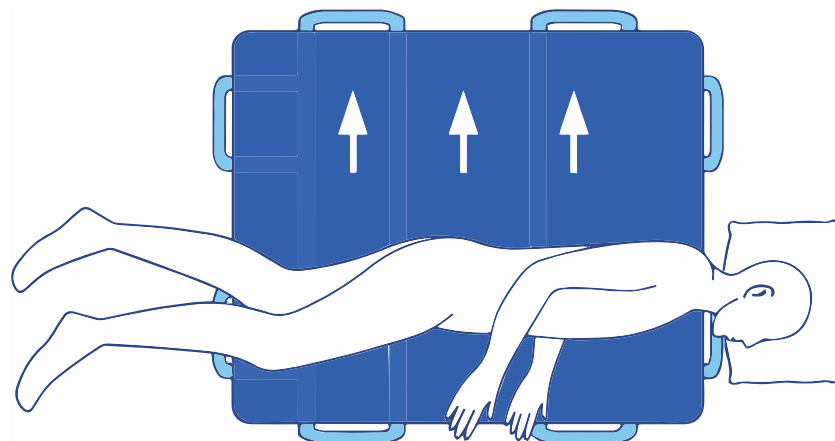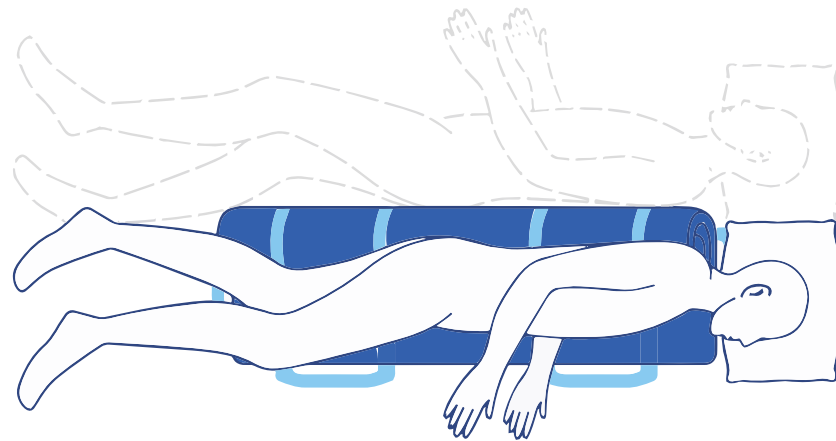

● METHODS OF USE:

1. To lift up the patient entirely for repositioning, it requires two people to work together and lift the side handles.

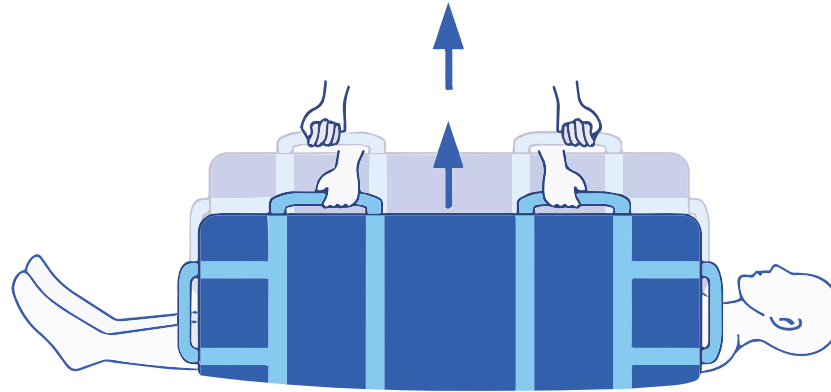

2. To lift the upper part of patient's body to maintain a sitting position, please lift the two upper handles.



● CAUTION:

1. This item is not 100% waterproof (water can seep through the webbing seam). If urinary incontinence occurs, please wipe it clean as soon as possible in case it soaks through bed sheets. If necessary, prepare a diaper on the positioning bed pad.

2. The positioning bed pad is smooth and should be used on a level bed surface. If you are going to raise an adjustable bed, for safety reasons, we suggest first removing the positioning bed pad in case slipping occurs.